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en PARI NaCl inhalation solution (0.9%) for moistening the respiratory tract

For your information

Content: 2.5 ml isotonic saline solution (sodium chloride) - sterile

Area of use: The PARI NaCl inhalation solution is a natural, isotonic saline solution (0.9%) in a concentration adapted for compatibility with human body fluids. It can be used to moisten the respiratory tract and as secondary treatment for cold-like symptoms such as coughing or nasal congestion.

When used according to the instructions, the PARI NaCl inhalation solution may be administered to infants, people of advanced years and to pregnant and nursing women.

Administration: Inhalation with jet nebuliser (e.g. PARI inhalation device) or ultra-sonic nebuliser.

Dosage: Up to 4 times a day with 2.5 ml of inhalation solution as required.

Warnings:

- Follow the device manufacturer's instructions for use.
- Discard after expiry date.
- Since the solution contains no preservatives, opened ampoules must be disposed of immediately for reasons of hygienic safety.
- This product includes small parts, which can pose a choking hazard. Always keep this product out of the reach of children aged three or younger!

Isotonic saline solution can cause reversible constriction of the bronchia when inhaled by very sensitive patients with bronchial asthma or a hyperreactive bronchial system. If this does occur, stop treatment and seek advice from your doctor or pharmacist.

Administration form and pack sizes:

20 × 2.5 ml; 60 × 2.5 ml; 120 × 2.5 ml

Application temperature: Room temperature

Explanation of symbols:

	Warning		STERILE A Symbol for sterility
	Production lot		Symbol for single use
	Expiry date		Please follow the instructions for use
	Manufacturer		

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PARI GmbH